


Oxygen Emergency Set

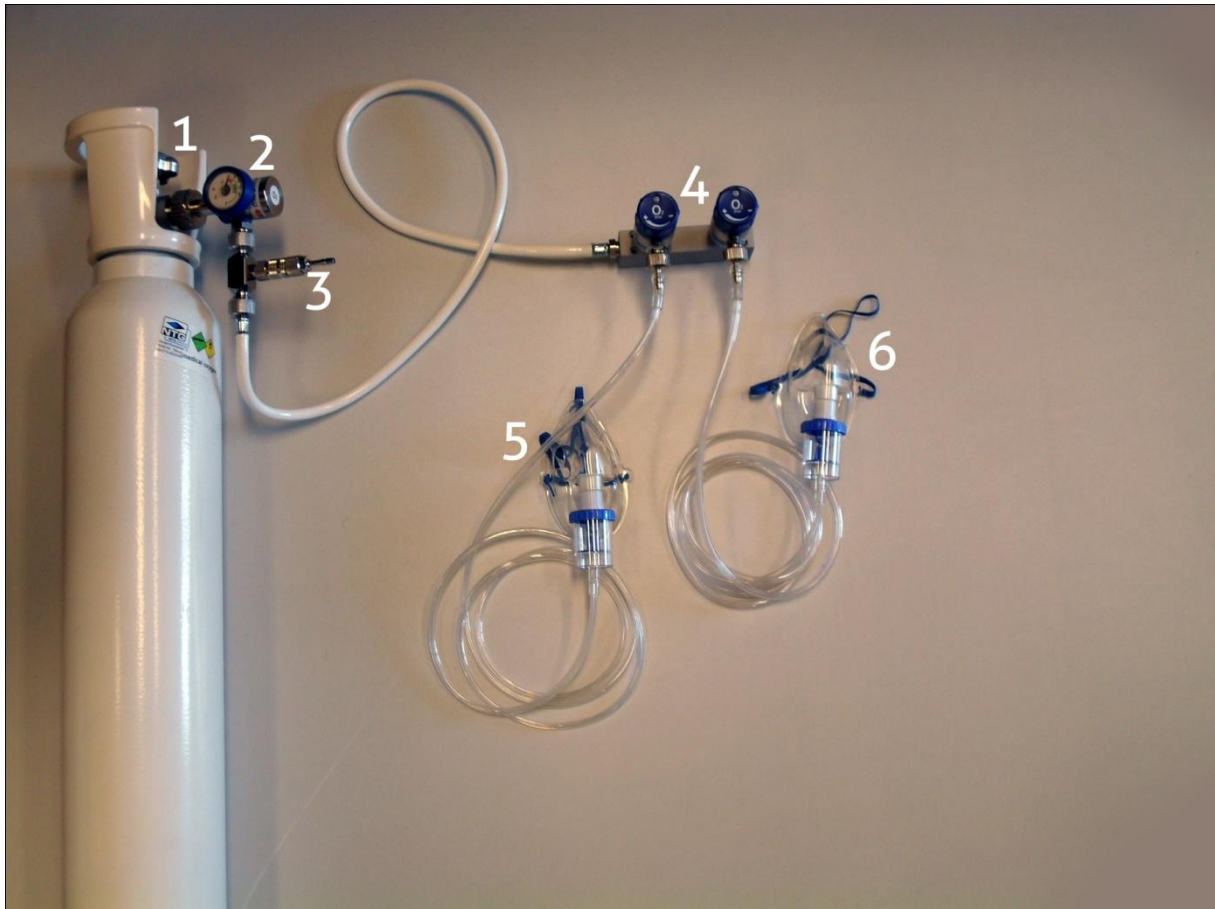

Important: Follow local guidelines for resuscitation.

Description:

- A: Choose the appropriate mask and attach this to the patient(s). (5+6)
- B: Adjust the flow (s) to be needed over the flow meters in Litres/min. (4)
- C: Open the shut off valve on the top of the oxygen cylinder very slowly. (1)
- D: Ensure that there is still oxygen in the cylinder
over the pressure gauge on the regulator. (2)
- E: To connect an external source (appr.5 Bar, >50 L/min O₂), use the probe
(connected to a flexible hose) and put it into the T-piece Quick connector
below the regulator. (3)

*Note: Due to recommendations of the manufacturer we prescribe to maintain and check this device each 5 years (see CE label on regulator).
The oxygen bottle must be checked every 10 years.
The medical oxygen has a shelf life of 3 years.*